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## GB

Treatment and prevention of ear wax plugs.

Spray solution.

## PROPERTIES:

Helps to break up and remove earwax plugs in the external ear canal.

Not greasy - Rinse-free.

The nozzle contains an antibacterial agent that insure best hygienic conditions.

## INDICATIONS:

• Hygiene of the external auditory canal, particularly for people with significant wax secretion.

• Those wearing a hearing aid.

• Can be used on infants over 6 months, children and adults.

## PRECAUTIONS FOR USE:

Do not use in the following instances: otitis, perforation to tympanic membrane, wearers of tympanic drains, or after middle ear surgery.

## STORAGE:

Store away from light at a temperature below 30°C.

## COMPOSITION:

Sodium docusatol: 0.5 mg

Preservative, Stabilisers, Purified water.

## INSTRUCTIONS FOR USE:

Place the nozzle at the entry of the auditory canal and press on the pump.

To remove an ear wax plug:

1 to 2 sprays and leave the product to act for 15 to 30 minutes.

For care: use 1 to 2 times per week in each ear.

Clean the entry of the ear canal with a cotton wool bud or the corner of a towel.

Rinse the nozzle in hot water after every use.

For additional hygiene, it is recommended also using Quies Docuspray ear spray with trace elements and Otoposon double-action sticks, to avoid pushing impurities inside the ear canal. Quies, a complete range of ear care for all the family.

30 ml spray bottle.

Dosing pump (approx. 150 doses) and nozzle containing an anti-bacterial agent.

Medical device compliant with European Directive 93/42/EEC

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