# DocuSpray

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### FR

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#### EAR SPRAY WITH TRACE ELEMENTS

##### Ear care

The ear is a delicate sensory organ. Cerumen secreted by the ear canal naturally protects the ear.

If the ear wax is too abundant or too thick, it is not naturally evacuated, it accumulates, gradually hardens and loses its ability to protect the ears, which then become more vulnerable to infections. Sometimes the ear wax can form a plug, particularly in the event of inappropriate use of cotton buds, which tend to force the wax back to the bottom of the canal. In order to keep the ears clean and healthy, regular hygiene of the ears is therefore necessary. DocuSpray ear spray is propellant gas free to avoid damaging the ear canal or eardrum.

DocuSpray ear spray is a trace-element based solution of marine origin for regular and delicate ear care. It contains active ingredients to break up ear wax and remove it. DocuSpray is suitable for the whole family. The bottle has two separate washable nozzles.

### INSTRUCTIONS FOR USE

**To ensure correct use, do not remove the safety cap.**

Straighten the arm and place it at the entry of the auditory canal. For each ear, conduct:

- 1 to 2 sprays 2 to 3 times per week.
- 1 to 2 sprays every day in the event of particularly dirty or dusty environment, frequent swimming, a dry atmosphere, or if you wear a hearing aid (remove the aid before spraying).

Prior cleansing of the ear with Quies Otospoon double action sticks facilitates the use of DocuSpray. For ideal hygiene, it is recommended using Doculyse once a month, a cerumenolytic, from the Quies ear care range.

##### PRECAUTIONS FOR USE

Rinse the nozzle with hot water after each use. Do not use in the event of otitis, perforated eardrum or a tympanostomy tube (ventilation tube).

If in doubt, consult your physician. Manufactured in a sterile environment, it is suitable for children over the age of 30 months. Keep out of the reach of children. For external use only.

##### COMPOSITION

Solution based on marine trace elements, non-ionic tensio-active agent, sodium bicarbonate, benzalkonium chloride, purified water.

Medical device compliant with European Directive 93/42/EEC

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