IT

Quiies

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Quiies

Protections auditives jetables en cire naturelle
Disposable protective earplugs made of natural wax

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Quies

Disposable protective earplugs made of natural wax

In order to maximize the efficiency of your earplugs, we should like to give you the following advice :

- Remove the cotton surround, then mould and warm up the earplug between the fingers, ensuring that your hands are perfectly clean.
- Rolling between the fingers, adapt the shape of the entire earplug to suit the size and shape of your auditory canal.
- Lightly insert the earplug in the ear. Pull the auricle back to release any air inside the auditory canal, in order to avoid buzzing noises in the ears.

QUIES earplugs will offer you constant protection against noise, water and dust pollution.

- Nominal size : 14

Recommendations

- It is essential to follow the instructions for use carefully.
- For health reasons, it is essential to wear the earplug constantly in noisy environments.
- We recommended you keep your QUIES earplugs in their box prior to use and store them in a dry place with a temperature below 25°C.
- **Single use only is recommended for QUIES earplugs.**
- QUIES earplugs are subjected to rigorous quality control tests and are hypoallergenic.

Warning

For optimum performance, please comply with the above recommendations for use.

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